Aerosol a pistone

Aerosol piston

Aerosol a piston

Aerossol de pistao

Kolbengerat fur Aerosoltherapie

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Pistonlu Aerosol

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EN PISTON AEROSOL UNIT

Aerosol therapy is indicated for treating inflammation of the upper respiratory tract such due to colds and nasal congestion as well as inflammation of the lower respiratory tract (bronchi and lungs).

The Chicco piston aerosol unit nebulises all types of medications quickly and effectively and the nebulised particles are ideal in size which ensures the deep and targeted deposit of medications, maximising the effectiveness of the therapy.

The Sidestream phial shortens medication administration time by increasing the flow of nebulised particles, making therapy easier and more convenient for children.

It is a class IIa medical device which converts medications from liquid to aerosol form, so they can be administered directly to the respiratory system.

CONTENTS OF PACKAGE

- 1- ON/OFF button
- 2- Compressed air outlet
- 3 - Filter
- 4 - Phial support
- 5- Carrying handle
- 6- Cable storage compartment
- 7- Openings for cooling air
- 8- Cable connection to mains
- 9 - Mask for children
- 10 - Mask for adults
- 11 - Nozzle
- 12 - Nose fork
- 13 - Phial
- 14 - Phial connection
- 15 - Connection tube

IMPORTANT WARNINGS

Before using this unit, carefully read the information included in this manual and keep it for future reference.

- Aerosol therapy unit always follow the instructions provided by your doctor regarding the type of medicine to use, dosage, and frequency and duration of the inhalation. Any use of the unit for uses other than for that which it was intended is considered improper and therefore dangerous; Artsana cannot be held liable for damages caused by improper, incorrect and / or unreasonable use of the unit or in cases where the unit is used in electrical systems not compliant with safety regulations.
- A power outage, a sudden malfunction or any other

adverse condition may cause the unit to stop working; therefore, it is recommended that users equip themselves with an alternate unit or type of medicine (agreed upon with a doctor) that can be used in the event this occurs.

• As soon as the unit has been removed from its packaging, ensure that it is intact and that there is no visible damage that may have been inflicted during transport. When in doubt, do not use the unit and contact your Artsana S.p.A dealer or distributor.

• NEVER DISASSEMBLE THE UNIT. All technical interventions must be carried out by Artsana-authorized personnel. For information contact your Artsana S.p.A dealer or distributor.

• Keep all packaging materials (plastic bags, boxes, etc.) out of the reach of children as they may be dangerous.

• Before connecting the unit make sure the information on the data plate are compatible with your electricity distribution network. This information is located on the bottom of the unit.

• In case of incompatibility between the unit's plug and the electrical socket, use adapters that have been certified under existing legislation in the individual countries or have an Artsana SpA authorised technician replace the plug.

• To prevent overheating and damage to the compressor, switch off the unit for at least 40 minutes after every 20 minutes of continuous use.

• To ensure correct operation, make sure that the air filter is dry.

• Do not use the unit in the presence of nitrous oxide, oxygen or anaesthetic mixtures flammable with air.

• Switch off and disconnect the unit from the mains after every use and before adding another type of medicine. Do not excessively fill the phial.

• Keep the unit and the power cable away from hot surfaces.

• Do not use the unit while taking a shower or a bath or in any humid environment, near bathtubs, sinks, faucets etc.

• Never touch the unit with wet or humid hands.

• Never drop or immerse the unit in water or other liquids. Should this occur, immediately disconnect the plug, stop use of the unit and consult technically qualified personnel.

• Should the unit be wet accidentally, immediately disconnect the plug, carefully dry the unit and if there is the possibility that the liquid has penetrated inside the unit, contact your Artsana S.p.A dealer or distributor.

• Do not block the ventilation openings during use.

• Do not use the unit if you feel tired or drowsy.

• If children or persons with physical or mental disabilities should use the unit, proper supervision is required. The unit has small parts which can become detached and be accidentally ingested. Children must be supervised to ensure that they do not play with the unit.

• this product is not a toy.

• this product must be assembled, maintained and handled by an adult.

• this product must remain in the place of use only for the time required for therapy. Once therapy has been completed, the unit must be stored in a safe place out of reach of children.

• Do not use fittings and accessories not recommended by the manufacturer.

• During use, make sure the unit is placed on a flat and stable surface to prevent any spillage. During use, the surface must be free of any objects that may obstruct the correct flow of air. Only use when in the horizontal position, as shown in figure A.

• Never leave the cable plug inserted in the electrical socket when the unit is not being used or when it is unattended.

• Before cleaning or performing any maintenance work on the unit, disconnect the unit from the mains power by remove the plug from the electrical socket.

• Should the unit no longer be used, after removing the plug from the socket, it must be made inoperative by cutting the power cable and disposing of the unit and the cable. It is also recommended that dangerous parts, especially for children, be rendered harmless.

• Refer to laws in force for information regarding the disposal of accessories subject to wear. To dispose of the unit itself, refer to EU law, 2012/19/EU.

• Do not use the unit if, after a fall, it shows signs of damage on any of its parts. When in doubt contact your Artsana S.p.A dealer or distributor

• In the event of a failure and/or poor operation of the unit, switch it off, removing the plug from the electrical socket and consult your Artsana S.p.A dealer or distributor. Do not attempt to open or tamper with the unit.

INSTRUCTIONS FOR USE

1. Position the unit horizontally on a flat, stable surface as shown in fig. A.

2. When you use the unit for the first time, clean the accessories indicated in the section "Cleaning the unit and accessories"

3. Unscrew the upper part of the phial (13) clockwise and insert the medicine into the reservoir, respecting the doses recommended by your doctor or pharmacist. Close the phial, screwing it clockwise back in

place (Fig. B);

4. Connect the tube (15) to the phial (Fig. C) and to the compressed air nozzle (2) (Fig. E);

5. Connect the phial (13) to the connection (14) and to the accessory you wish to use for therapy: fork, mouthpiece, mask for children or adults (Fig D);

6. Plug the unit into an electrical socket;

7. Turn on the unit using the switch(1);

8. Start the inhaling treatment. During therapy the phial can be rested on its dedicated support (4);

9. Once therapy is complete, switch off the unit and remove the plug from the electrical socket. Put the cable back into the dedicated compartment (Fig. F);

10. When the aerosol flow becomes intermittent, stop treatment for a few seconds to allow the suspended drops of solution to fall on the walls of the phial. Resume the treatment session, which will be finished when no more nebulised solution is issued from the phial;

11. Clean the unit and accessories as indicated below.

CLEANING THE UNIT AND ACCESSORIES

Before cleaning or carrying out maintenance work, always switch off the unit and remove the plug from the electrical socket. It is recommended the unit be cleaned immediately following treatment is complete.

Cleaning and Maintenance of the Unit

Warning! Never immerse or soak the unit in water or other liquids. For cleaning, use only a clean, dry cloth.

Cleaning and maintenance of the phial and accessories

Completely disassemble and separate all parts of the phial and soak them in boiling water for about 5 minutes. Warning! Only the phial can be boiled.

Rinse all accessories under running water. Use denatured alcohol to disinfect the parts which come into contact with the patient. Carefully dry all parts and the unit before storing it. Store the appliance in a cool, dry place, away from sunlight and heat. Never use benzene, thinners or any flammable chemical substance to clean the unit. To ensure greater hygiene when the unit is used by several people or in case of professional use (in clinics, hospitals, etc.) or rented use, each person should use their own kit.

For information about accessory replacement kits, contact your local Artsana S.p.A dealer or distributor.

CHECKING AND REPLACING FILTERS

The device is equipped with a filter (3) that protects the compressor from dust infiltration.

Periodically check the condition of the filter. Correct filter maintenance prolongs the life of the aerosol. The frequency with which filter should be replaced depends on the conditions in which the unit is used. However, it is recommended that filters be replaced after 20 working hours or in the event they are visibly dirty. The filter is located on the lower part of the aerosol unit. To replace the filter:

- Open the filter cover cap, using a small pointed object if necessary
- Remove the filter to be replaced from its housing; use a sharp-pointed object for assistance if necessary;
- Carefully insert the new filter;
- Return the filter cover cap to its original position

TECHNICAL SPECIFICATIONS

Nominal voltage: 230 V

Frequency: 50 Hz

Power: 130 VA

Fuse: T 1.6 AL 250V ~

Maximum flow: 15 ± 2 l/min.

Operational flow: 7 ± 1 l/min.

Max pressure: $2 \pm 0,4$ atm

Operating pressure: $0,6 \pm 0,1$ atm

Protection against liquids: IPX0

Weight: 1210 g

Device conditions of use: 20' ON - 40' OFF

MMAD: $3,2 \pm 0,2$ μ m

Breathable fraction: $72,2 \% \pm 2\%$

Phial capacity: 5 ml

Noise level at 50cm*: 63 dB(A)

nebulisation speed: $0,25 \pm 0,04$ ml/min.

residual volume not nebulised: $0,8 \pm 0,05$ g

% < 5μ m: $72,2\% \pm 2\%$

% between 2 and 5 microns 51,4%

*The data relative to noise were measured with a new unit. Values may vary with use.

If the product is used and maintained in accordance with the instructions for use, it should have a life expectancy of at least 1000 operating cycles. In the event of breakage or damage, dispose of according to instructions.

The information on the unit's capabilities provided by the manufacturer in accordance with EN 13544-1 may not apply to suspended or viscous medicines. EN 60601-1 Electrical-medical devices -Part 1: general safety standards

EN 60601-1-2 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral standard: Electromagnetic compatibility - Requirements and tests

EN 13544-1 Respiratory therapy units - Part 1: nebulisation systems and related components

SYMBOLS KEY

WARNING!

Consult the Instructions for Use

Type BF applied part

Double insulation

Compliant with MDD Directive 93/42 EEC + 2007/47/EC

Alternate current

Storage and transport conditions

Humidity: 15%/85%. Store in a cool, dry place

Temperature: -10°C/+55°C

Environmental Conditions of Use:

Humidity: 15%/85%
Store in a cool, dry place

Temperature: 10°C/40°C

The medical device requires special precautions regarding EMC and must be installed and used according to the information provided in the accompanying documents.

WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use.

The warranty shall not therefore apply in the case of damages caused by improper use, wear or accidental events.

For the duration of warranty on conformity defects please refer to the specific provisions of national laws applicable in the country of purchase, where provided.

Manufacturer's Handbook and Declaration – Electromagnetic Emissions		
The Chicco piston aerosol can be used in the electromagnetic environment specified below. The customer or user of the aerosol unit must ensure it is used in such an environment		
Emissions Tests	Compliance	Guide to electromagnetic environments
CISPR 11 Radiated/Conducted Emissions	Group 1	The Chicco piston aerosol only uses RF energy for its internal functions. Therefore its RF emissions are very low and do not cause any interference when in proximity to electronic devices.
CISPR 11 Radiated/Conducted Emissions	Class [B]	The Chicco piston aerosol is suited for use in all environments, including those domestic and those directly connected to the public electricity distribution line that provides power to buildings used for domestic purposes.
Harmonics IEC/EN 61000-3-2	Class [A]	
Voltage fluctuations/flickers IEC/EN61000-3-3	[Compliant]	

Manufacturer's guide and declaration – Electromagnetic Immunity		
The Chicco piston aerosol can be used in the electromagnetic environment specified below. The customer or user of the aerosol unit must ensure it is used in such an environment.		
Emissions Tests	Compliance	Guide to electromagnetic environments
Electrostatic discharges (ESD) IEC/EN61000-4-2	±6kV on contact ±8kV in air	Floors should be made of wood, cement or ceramic. If the floors are covered with synthetic material, the relative humidity should be now higher than 30%
Fast/burst transients IEC/EN61000-4-4	±2kV power	The power supply should be that typical of a commercial building or hospital.
Surge IEC/EN61000-4-5	±1kV differential mode	The power supply should be that typical of a commercial building or hospital.
Voltage dips, brief interruptions and variations in voltage IEC/EN61000-4-11	<5%UT for 0.5 cycle 40%UT for 05 cycle 70%UT for 25 cycle <5%UT for 5 sec	The power supply should be that typical of a commercial building or hospital. If the aerosol user requires that the unit operate continuously, it is recommended that it be used with a UPS device.
Magnetic field IEC/EN61000-4-8	3A/m	The magnetic field should be that typical of a commercial building or hospital.
Conducted Immunity IEC/EN61000-4-6	3Vrms 150kHz to 80MHz (for non-life supporting devices)	
Radiated Immunity IEC/EN61000-4-3	3V/m 80MHz to 2.5GHz (for non-life support devices)	
Note: UT is the value of the power voltage		

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(EN) THIS PRODUCT COMPLIES WITH THE DIRECTIVE 2012/19/EU.

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the disused appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both the environment and health, and contributes towards the recycling of the product's materials. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

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(TR) BU ÜRÜN 2012/19/AB DİREKTİFİNE UYGUNDUR.
Artsana S.p.A. - Via Saldarini Catelli 1 - 22070 Grandate (CO) Italy