# aerosleep®

## OYO

Monitor

## CONTENT

EN	6
NL	20
FR	34
DE	48
DA	62
ES	76
GR	90
IT	104
NO	118
SE	132

Hello!

## LET'S INTRODUCE

OYO is a smart device designed to help you monitor your baby.

It tracks:

- Breathing movement while sleeping
- Skin temperature
- Body position
- Activity level

OYO connects to your smartphone with the **OYO smart baby monitor** App, where you can check these statistics real-time and a history of the last 72 hours.

OYO also notifies you through your smartphone as soon as:

- low breathing movement was detected
- your baby's skin temperature is rising
- your baby rolls on the belly
- a fall was detected
- your baby is quiet or active
- your baby makes noise (only when combined with the OYO Camera, available separately)

### OYO smart baby monitor App

The Camera uses the same **OYO smart baby monitor** App as the OYO Monitor. Available in the App Store and in Google Play.

### WHAT'S IN THE BOX?

- OYO Monitor
- OYO docking station
- Belly band
- USB cable
- Wall charger
- Manual

**Let's get started!**

## Charge your OYO Monitor

Place the OYO Monitor on top of the OYO docking station with the sensor facing down, and plug in the wall charger.

The light on the OYO docking station will start blinking green and the Monitor will charge automatically.

Leave the Monitor for about 1 hour.

## Download the OYO smart baby monitor App


Download the **OYO smart baby monitor** App on your smartphone and personalize your baby's settings. Follow the instructions on the screen.


The **OYO smart baby monitor** App is available in the App Store and in Google Play.

Minimum requirements  
smartphone, scan QR code:

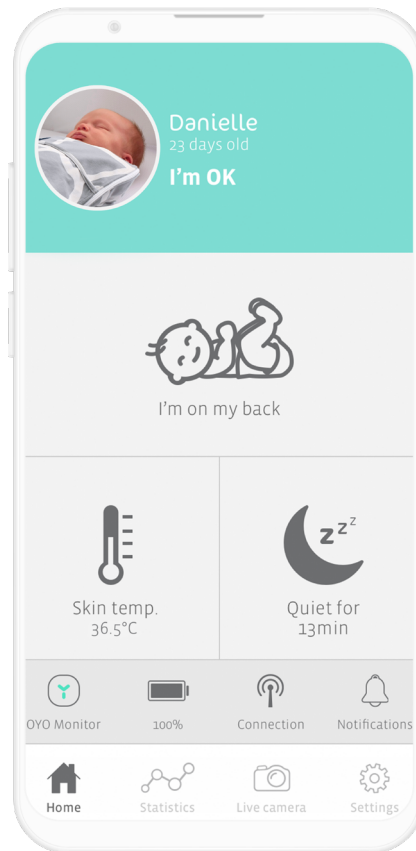# OYO connections
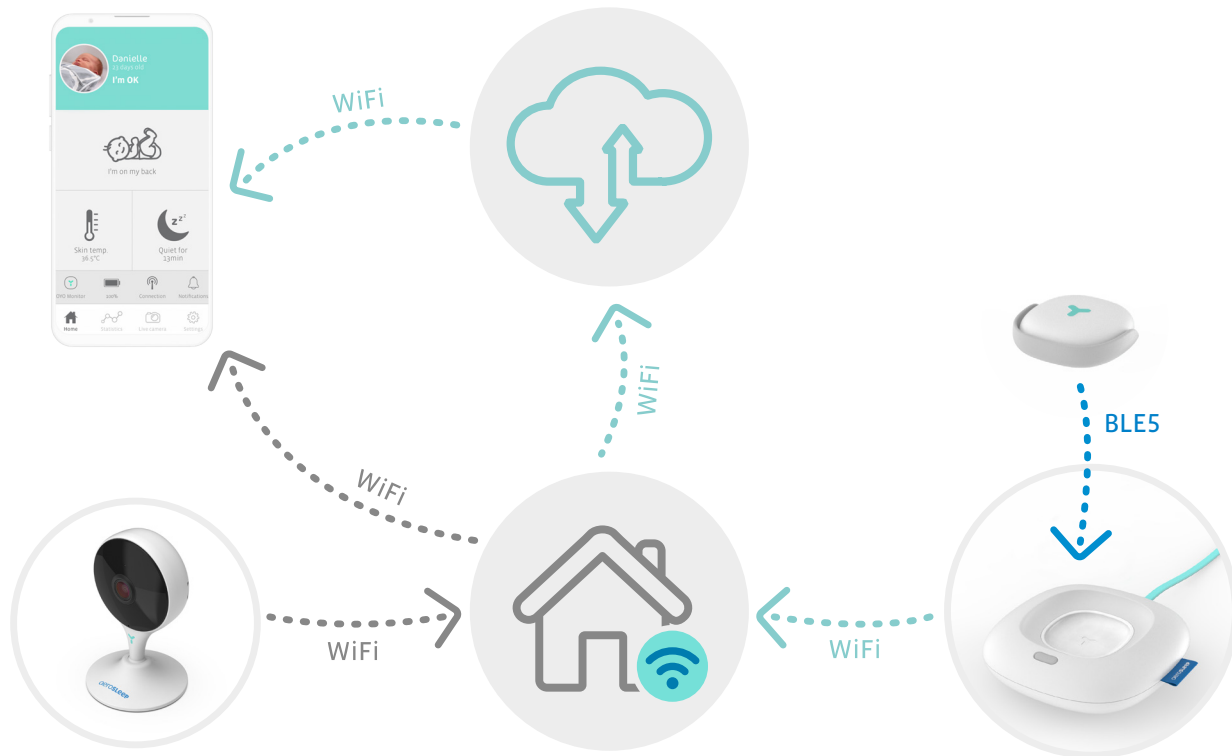
## Attach the OYO Monitor

### step 1

Open the belly band and place it under your baby with the 'Attach OYO here' ribbon facing up.

### step 2

Open the Monitor's grey hinge and attach it onto the ribbon with the sensor facing up. Ensure the hinge is properly closed.

### step 3

Wrap the right-hand side of the belly band over your baby's belly. Make sure the Monitor is in direct contact with your baby's skin.

### step 4

Close the other side of the belly band over your baby's belly so it fits snugly.

To ensure accurate tracking, please make sure the OYO Monitor is in constant contact with your baby's skin.

When you are not using the OYO Monitor on your baby, please make sure you deactivate the Monitor in the OYO App or by placing it on top of the OYO docking station.

## How to activate the OYO Monitor

After attaching the Monitor to your baby's belly band, short-press the button on the OYO docking station.

The docking station's light will flash 'red – yellow – blue' for 1 second.

Alternatively, you can enable or disable the Monitor in the OYO App.

The OYO Monitor is now ready to monitor your baby!

After about 5 seconds, you should see the status in the App change to 'I'm OK' and the light of the OYO docking station will switch off.

If the docking station is enabled as night light, it will show a white light.

The OYO Monitor connects to your smartphone through the OYO docking station. Please make sure your OYO docking station is always plugged in when using the OYO Monitor.

### Are you no longer using the OYO Monitor on your baby?

Detach the OYO Monitor from your baby's belly band and place it back on your OYO docking station with the sensor facing down.

The Monitor will deactivate and start charging automatically. Alternatively, you can disable the Monitor in the OYO App.

## How do I know my baby is OK?

The OYO App notifies you of certain changes in your baby's situation by making a continuous or short noise, depending on the nature of the change. Always check your App when you receive a notification to see if your baby needs any help.

These notifications can also be disabled or enabled in the App settings.

Notification	short, once	loud, continuous
No breathing movement detected		Y
Low breathing movement detected		Y
Skin temperature is high		Y
Fall detected		Y
Skin temperature is rising	Y	
Your baby rolled on its belly	Y	
OYO Monitor lost connection	Y	
Your baby is quiet	Y	
Your baby is active	Y	
Your baby is making sounds*	Y	
OYO Monitor needs charging	Y	

\*only when used in combination with OYO Camera

## What does the light on my OYO docking station mean?

colour	function	status	OYO docking station	OYO Monitor
yellow	network issues	blinking	no internet connection	no Bluetooth connection
green	confirmation network connection	blinking	ready for pairing	ready for pairing
red	baby notification	constant		notifications
blue	charging states	blinking		charging now
		constant		fully charged
multicolor	confirmation action	red/yellow/ blue for 1 sec	After short-pressing (1 sec) the OYO docking station button to activate the Monitor After long-pressing the OYO docking station button to reset	
white	night light on (in settings)	constant	connected	connected
off	night light off (in settings)		connected	connected

## Troubleshooting

If the answer to your question is not in this list, our experts are happy to help: [hello@aerosleep.com](mailto:hello@aerosleep.com)

Issue	What you see in the App	Possible causes	Advice to solve the problem
<b>OYO Monitor disconnected</b>	‘OYO Monitor disconnected’	<b>1.</b> Battery not charged	Check if the Monitor battery is charged
		<b>2.</b> Monitor out of range	Check if the OYO Monitor & docking station are in the same room
<b>OYO docking station disconnected</b>	‘Dock disconnected’	<b>1.</b> Internet issues	Check if your Wi-Fi network is working and set to 2,4 GHz. Check if the OYO docking station is within reach of the router. If not, move it closer.
		<b>2.</b> Not plugged in	Check if your OYO docking station is plugged in.
		<b>3.</b> Docking station connected to different Wi-Fi network	Go to ‘Settings’ in the OYO App, tap your baby’s name and then ‘Wi-Fi settings’. Connect the OYO docking station to your Wi-Fi network.
<b>Phone disconnected to internet</b>	‘No internet connection’	Phone has no internet or Wi-Fi connection	Make sure your smartphone is connected to internet

Please ensure that your Wi-Fi router is set to 2,4 GHz.

Issue	What you see in the App	Possible causes	Advice to solve the problem
<b>Installation issue:</b> something went wrong while trying to connect OYO to your App account	'Something went wrong'	<b>1.</b> OYO docking station not plugged in	Check if the OYO docking station is plugged in.
		<b>2.</b> OYO device already linked to baby account	If you're not the first one to install this device, the Monitor could still be linked to a different baby account. In this case you should reset the OYO docking station by pressing and holding the button for 10 sec. Then retry to connect. If you would like to have access to the same baby account, ask the person that installed this account to share this device with you (in the App settings).
<b>Installation issue:</b> something went wrong while trying to connect	'Something went wrong'	<b>1.</b> Internet issues	Check if your Wi-Fi network is working and set to 2,4 GHz.
		<b>2.</b> Wrong Wi-Fi password	Check your Wi-Fi password Reset the OYO docking station by pressing and holding the button for 10 sec, and retry to connect.

### How to reset?

Press and hold the button on the OYO docking station for 10 seconds until the light flashes red – yellow – blue for 1 sec.

**Important!**

- To ensure accurate tracking, the OYO docking station should be stationed in the same room as your baby when using the OYO Monitor.
- For safety reasons, the OYO docking station should stay out of your baby's reach.
- A good quality Wi-Fi connection (2,4 GHz) is required for proper functioning.
- To ensure accurate tracking, attach the OYO Monitor to your baby's snugly fitting belly band according to the instructions in this manual.
- OYO tracks skin temperature and not core body temperature, although the difference is very small. When placing the OYO Monitor on your baby's skin, the skin temperature slightly drops for a short period of time. That is why the Monitor needs time to stabilize, you may notice this in the App as well.
- OYO's skin temperature tracking is comparable to armpit temperature measurements. The result may vary up to  $\pm 0,0^{\circ}\text{C}$  from rectal temperature measurements.

- OYO is not a medical device and therefore not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap.
- OYO is designed to help you monitor your baby and is not to be used as a substitute for adult supervision.
- Fully charged, the OYO battery will last up to 48 hours of continuous use. Check the OYO App for your Monitor's battery level (on the main screen) and recharge the Monitor when the battery level is low.
- OYO can only track breathing movement when your baby is sleeping in a steady bed or cot. OYO is therefore not suitable for co-sleeping or bed sharing.

**Minimum requirements smartphone, scan QR code:**

## Disclaimer

Customers use the OYO Monitor at their own risk and QLEVR N.V., its members, employees, agents, contractors, representatives, manufacturers and associates shall not be held liable for any loss, damage or injury suffered by the customer or any other person, arising from or in connection with the use or manufacture of this device. This includes loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of OYO. All warranties and any other liability from whatsoever cause arising, are hereby excluded and the customer hereby indemnifies QLEVR N.V. in respect of any such liability. QLEVR N.V. guarantees this product to be free from defects in materials and workmanship for a period of 12 months from original date of purchase, verified by a sales receipt.

Electrical devices must be recycled or disposed of according to the applicable regulations.

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