Nuby
NATURAL TOUCH™

- **F** SET TIRE-LAIT MANUEL: MODE D'EMPLOI
- **D** MANUELLES MILCHPUMPEN-SET: BEDIENUNGSANLEITUNG
- **I** SET TIRALATTE MANUALE: MANUALE D'ISTRUZIONI
- **NL** MANUELE BORSTPOMP SET : GEBRUIKSAANWIJZING
- **ES** EXTRACTOR DE LECHE MANUAL : MANUAL DE INSTRUCCIONES
- **PT** CONJUNTO DE BOMBA DE EXTRACÇÃO DE LEITE MANUAL : MANUAL DE INSTRUÇÃO
- **SE** MANUAL BRÖST PUMP SET : BRUKSANVISNING
- **DA** MANUEL BRYST PUMPE SÆT : BETJENINGSVEJLEDNING
- **FI** MANUAALINEN RINTAPUMPPUSETTI : KÄYTTÖOHJEET
- **UK** MANUAL BREAST PUMP SET : INSTRUCTION MANUAL
- **GR** ΣΕΤ ΧΕΙΡΟΚΙΝΗΤΟ ΘΗΛΑΣΤ : ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
- **RU** РУЧНОЙ МОЛОКООТСОС В НАБОРЕ : РУКОВОДСТВО ПО ЭКСПЛУАТАЦИИ
- **KZ** ҚОЛ СҮТ СОРҒЫШЫ ЖИЙНТЫҚТА ПАЙДАЛАНУ НҮСҚАСЫ
- **AR** ةيودي ل ي دثلا ةخضم ةعوم جم : تاداشر ل ل ي د

**F** **CONTENU**

- D** INHALT
- I** INDICE
- NL** INHOUD

UK **CONTENT**

- EN** INNEHÅLL
- SE** SISÄLTÖ
- DA** INDHOLD

ES **CONTENIDO**

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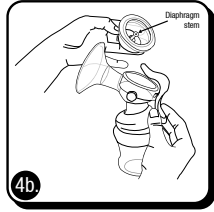
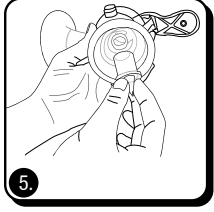
- UK**
- Begin by removing the Travel Cover from the Comfort™ funnel.
 - Remove the Silicone Insert™ from the Comfort™ Funnel by pulling side of insert over edge of funnel.
 - Lift the Pump Hood from your Breast Pump by pulling hood with a tilting motion.

- a+b. Remove the Vacuum Membrane from your Breast Pump by pulling up and sliding the Diaphragm stem out of slot in the Pump Lever.
- Remove the Flow Valve by pulling valve down and away from cylinder.

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UK Customize your Comfort™ Breast Pump with the Adjustable Vacuum Valve. The Nüby™ Comfort™ Breast Pump provides more flexibility and convenience when it comes to breast feeding. You can increase or decrease the suction pressure by simply turning the Adjustable Vacuum Valve! Note: While the Adjustable Vacuum Valve is removable, it is not necessary for cleaning and may render the pump inoperable if it becomes lost. Adjusts suction pressure for individual comfort.

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☑ Comfort™ Breast Pump

CLEANING AND STERILIZING YOUR Nüby™ Comfort™ BREAST PUMP

Start by disassembling all parts, then wash in the dishwasher, top rack. (keep small parts

07 - NTNV 72035

in a dishwasher basket to prevent loss). Or hand wash in soapy water and rinse with clean water. Sterilize your Comfort™ Breast Pump in the Nüby™ Microwave Steam Sterilizer or boil in water for 5 minutes.

RE-ASSEMBLING YOUR PUMP BEFORE USE

- Wash hands thoroughly. Replace the Flow Valve onto the cylinder located in the (throat) portion of the pump. Make sure this fits securely and high on the cylinder.
- Screw the 5oz./150ml. Silicone Nürser™ onto the bottom of your Nüby™ Breast Pump.
- Insert the Vacuum Membrane at the top of pump and slide Diaphragm stem into the slot at the front of Pump Lever. Push down and run finger around outside edge to secure the seal.
- Replace Pump Hood over Vacuum Membrane and push down.
- Fit the Silicone Insert inside the Comfort™ Funnel and pull edges over the lip of Funnel.

Your Nüby™ Comfort™ Breast Pump is now ready to use or, if you need to travel, just snap the Travel Cover on the front of the Pump Funnel and it will remain Clean and Sterile until ready for use.

WHEN TO USE YOUR Nüby™ BREAST PUMP.

- Two to four weeks after birth, your milk supply and feeding schedule should be well established. This would be a perfect time to begin getting familiar with the operation of your new breast pump.
- You can begin sooner or exceptions apply if:
 - You need to express milk for your baby while in the hospital.
 - Your breasts are engorged (painful and swollen), expressing some milk before or between baby's feeding times can ease discomfort and help with infant latch on.
 - You are separated from your baby and wish to continue to breast feed when you are reunited, expressing your milk regularly will stimulate your milk supply.
- Find the time of day that's best for you to express your milk, for instance:
 - early in the morning when your breasts are full or just before or after your baby's first feed, or
 - after a feed, if your baby has not emptied both breasts, or
 - between feeds, or during a break, if you have returned to work.

HELPFUL HINTS FOR SUCCESS

- Using your Nüby™ breast pump may take practice - it may take several attempts before success.
- Have your baby or a photograph of your baby nearby to help encourage "let-down".
- Place a warmed Nüby™ Breast Aid™ gel pad or warm cloth on the breast for a few minutes before you start pumping. This can assist milk flow and alleviate painful breasts.
- Practice. Find the technique that works best for you. Don't be discouraged if your milk doesn't flow immediately. Relax and continue. Start with only a few minutes at a time from each breast. If the process becomes painful, STOP and seek the advice from a lactation consultant.

HOW TO USE YOUR Nüby™ BREAST PUMP.

- Wash hands thoroughly.
- Make sure you have sterilized and carefully assembled your pump as indicated
- Again, look over the (HELPFUL HINTS FOR SUCCESS) section.
- Make yourself comfortable in a chair or sofa. Have a glass of water or other drink nearby. Lean slightly forward to apply pump to your breast.
- Press the funnel and silicone Insert of the pump firmly against your breast making sure that no air can escape to ensure suction
- Push down on the Pump Lever and you will begin to feel suction. A full stroke of the lever is not needed to create a vacuum - only as much as is comfortable. (The Adjustable Vacuum Valve will allow you to increase or decrease the suction. You can customize the setting according to your personal comfort).
- Milk will begin to flow after the first few pumps. Hold the vacuum for 2-3 seconds and ALWAYS ALLOW THE PUMP LEVER TO RETURN TO ITS RESTING POSITION AFTER EACH STROKE. This will keep milk from building up in the body of the pump. MAXIMUM SUCTION IS LESS IMPORTANT THAN YOUR PERSONAL COMFORT. YOUR NÜBY™ BREAST PUMP WILL GENERATE ENOUGH SUCTION WITHOUT MAKING A COMPLETE STROKE. UNLESS YOU FEEL THE NEED FOR ADDITIONAL STIMULATION, IT IS NOT NECESSARY TO CONTINUE PUMPING THE LEVER. THE SUCTION NOT THE PUMPING ACTION, WILL DRAW THE MILK. THE 2 TO 3 SECOND CYCLE DESCRIBED ABOVE IMITATES YOUR BABY'S NATURAL SUCK/RELEASE RHYTHM.
- While some women may prefer to use the pump without the silicone insert, studies have shown that the massaging effect of the Nüby™ Silicone Insert stimulates the areola and begins the "let-down" process more quickly.
- On average, it will take 10 minutes to express 60/120ml. or 2 to 4 ozs. of breast milk, however, this is just an estimate and will vary from woman to woman.

STORAGE AND USE OF YOUR EXPRESSED BREAST MILK.

- Without refrigerating, breast milk stays good up to 6 – 8 hours. Then it should be used. Do not store it anymore.
- In the fridge breast milk can be stored for up to 3 days (+/- 4°C, not in the fridge door). In the freezer at -18°C it can be stored up to 3-4 months. In a separate freezer at -18°C it can be stored up to 6 months. While transporting the cooled or frozen milk the cooling process shouldn't be interrupted.
- Refrigerate or freeze immediately any breast milk that is to be stored.
- Milk can be stored in covered Nüby™ Nürsers using the Nüby™ storage caps. Clearly label each bottle with date of expression.
- Do not fill bottles completely, if they're to be frozen, allowing room for milk to expand while freezing.
- Breast milk can be heated in the same manner as other bottle feeds. (But never microwave as this may destroy valuable nutrients and antibodies).
- Frozen breast milk can be defrosted overnight in the refrigerator or heated gradually in warm water.
- NEVER immerse a frozen bottle in boiling water as this may cause the bottle to crack or split.
- Once thawed, use within 24 hours.
- Breast milk should NEVER be re-frozen.

11. Heated breast milk left-overs should be discarded.
12. DO NOT add fresh breast milk to already frozen or refrigerated milk for storage.
13. When you've expressed milk within 24 hours then the breast milk – if cooled – can be mixed with other cooled breast milk.
14. During the storage of milk it is possible that it gets a watery layer with a greasy layer on top. By gently stirring it should get back to milk substance. Also while freezing it is possible that the breast milk slightly changes colour. All this hasn't got any influence on the quality of the breast milk.
15. If the breast milk is stored in a bottle, to feed baby, be sure to use a fresh sterilized nipple.

Note: The methods for storage and use of breast milk mentioned in this booklet are intended to serve as only a guide and should not be taken as the definitive methods. Please consult your physician or your lactation consultant for more information concerning proper methods for storage and use Page 9

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UK PP BOTTLE

BOTTLE: Made from unbreakable polypropylene. Can be boiled; top rack dishwasher safe; microwave safe for warming fluid only. Always warm fluid with cap and nipple removed from bottle. Graduated in ml. **NIPPLE**: Made of soft silicone. Can be boiled. With your baby's comfort in mind, an anti-colic feature was designed into the nipple. This helps to prevent ingestion of air by baby. (ingestion may cause colic). Nüby™ has taken this one step further by adding the new Nüby™ nipple with raised offset surfaces that assist in the eruption of teeth by gently massaging infant's gums. For your child's safety and health; **WARNING**: This product must always be used under adult supervision. Accidents have occurred when babies have been left alone with drinking equipment due to the baby falling or if the product has disassembled. Please remove hood from bottle before use and keep it out of reach of children. Do NOT let a child take a bottle to bed or self-feed for long periods. Continuous and prolonged sucking of fluids will cause tooth decay. Tooth decay in young children can occur even when non-sweetened fluids are used. This can occur if the baby is allowed to use the bottle/cup for long periods through the day and particularly the night, when saliva flow is reduced or if it is used as a soother. Ask your doctor for more information about proper bottle feeding. **INSTRUCTIONS OF USE**: Before first use place in boiling water for 5 minutes. This is to ensure hygiene. Always check food temperature before feeding. Never use with carbonated beverages, as carbonation places product under pressure and may cause leakage. Test the nipples before each use by pulling in all directions. Discard nipples from the moment they start showing signs of damage (enlargement, cracks). **CAUTION**: Do not expose nipples to sunbeams or other heat sources. Do not leave nipples in a disinfectant product for longer than necessary (follow instructions on disinfectant). Do not boil in microwave. Burns and scalding may result if bottle is overheated in microwave. **CLEANING**: Clean and sterilize bottles and nipples

NTNV 72035 - 14

before and after each use. Wash in dishwasher (top rack only) or in warm water with a mild soap. Rinse thoroughly. DO NOT sterilize bottles in sterilizer that is only made for glass bottles. Do not leave nipples in a disinfectant product for longer than necessary (follow instructions on disinfectant).

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UK

GUARANTEE

Keep the receipt or invoice of this product !

- New Valmar BVBA guarantees its products for the legal period of time starting from the date of purchase. If any defect due to faulty materials and workmanship occurs within the guarantee period New Valmar BVBA will repair or replace at its expense. New Valmar BVBA will only pay for repair or replacement if convincing proof is provided (by means of a receipt or invoice) that the day on which service is claimed is within the guarantee period.
- The guarantee does not cover products and/or product parts that are subject to wear, that can be considered as consumable parts by their nature or that are made by glass.
- The guarantee is not valid if a defect is due to damage caused by incorrect use, poor maintenance (e.g. parts blocked by scale) or if alterations or repairs have been carried out by persons not authorized by New Valmar BVBA.
- For the appliance to be used correctly, the user should strictly adhere to all instructions included in the instructions for use and should abstain from any actions or uses that are described as undesired or which are warned against in the instructions for use.
- These guarantee restrictions do not affect your statutory rights.
- Take your defective product - with the receipt or invoice of this product - back to the shop where you purchased this product.

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