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Conforms to European standards in force

**Utilization precautions :**

- Carefully read all the instructions before using the sterilizer.
- Before using the device for the first time, check that your electrical installation is compatible with the device's rating.
- If any defect is visible on the device, the plug or the power cable, disconnect the device and do not use it.
- Always place the sterilizer on a flat dry surface outside the reach of children. Remember that nothing replaces the watchfulness of parents.
- Do not move the device when it is operating or when it contains hot liquid.
- Do not touch hot surfaces.
- The steel heating base becomes very hot and the steam released by the device can cause serious burns.
- Never immerse the device in water. Never immerse the power cable, the plug or the steel heating base in water or in any other liquid.
- This device must not contain any accessory inside it. Do not dismount the device.
- Do not place anything on this device.
- This device is strictly designed for household usage. It must never be used outdoors.
- If the power cable has to be replaced, it must be done by the manufacturer or his agent to avoid any danger.
- This product must not be used by children. Keep the sterilizer and its cable out of the reach of children. This product may be used by people with reduced physical, sensory or mental capacities, or with insufficient understanding of the product, but only under supervision or if they have been given instructions on how to safely use the product and are fully aware of all potential dangers

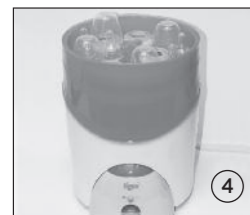
- This product is intended for use in the home or in equivalent environments, such as: kitchen spaces reserved for staff in shops, offices and other professional environments; farms; use by guests in hotels, motels and other residential type environments; bed and breakfast type environments.
- Watch children to make sure they do not play with the device.
- When using the device, be careful because the device releases hot steam. Do not place the device on or near hot surfaces, in an oven or close to an electric or gas stove.
- Disconnect the device during refilling, cleaning, storage and when you do not use it.
- Keep this user guide handy.

#### How to use your sterilizer

Before putting products in the device to sterilize them, it is important that all baby bottles, nipples and accessories be empty and be washed and abundantly rinsed in cold water.

1. Place the sterilizer on a flat surface outside the reach of children where it cannot move while it is being used.
2. Remove the sterilizer from the steel heating base and remove the lid.
3. Place the rinsed baby bottles with the neck openings facing downward on the stack plate provided for this purpose (the stack plate is placed inside at the bottom of the tank).

4. Place the accessories basket on the tank.
5. Place the nipples and other accessories in the accessories basket.

6. Reinstall the lid. Make sure the lid is properly installed. The two arrows should be aligned.

7. Remove the doser integrated in the control console and fill it twice to obtain approximately 60 ml of cold water and pour it on the steel heating base.

8. Carefully reinstall the tank on the base ; make sure that the tank edges fit correctly on the steel heating base.
9. Connect the device to the wall outlet. (The device must be grounded).

10. Switch the sterilizer on by sliding the white button into the On position and then pressing the on/off switch.

**WARNING :** Be careful to avoid the hot steam that can exit through the ventilation hole on the lid.

11. The light indicator and the heating automatically turn off once sterilization is terminated after 6 minutes. To interrupt the sterilization process or switch off the sterilizer once the process is complete, slide the white button into the OFF position. It is strongly recommended to wait a few minutes before opening the lid so that the device and its contents can cool off. Then you can remove the baby bottles and the accessories.

**IMPORTANT :** Always wait for the sterilizer to cool off for at least 20 minutes before beginning a new sterilization.  
If you want to sterilize 8 standard neck baby bottles, place them directly on the tank bottom after removing the stack plate.

#### Maintenance recommendations :

**IMPORTANT :** Always disconnect your sterilizer before cleaning it.

- Use a moist cloth to clean the control console.
- The tank, the lid and the baby bottle support can be washed in your dishwasher and also cleaned with hot soapy water.
- After the first utilization, a brownish limestone deposit caused by the heat may form on the heating plate's surface. This deposit is not harmful, but can prevent the device from operating correctly if it stratifies.
- Avoid using anti-limestone capsules, chemical sterilizing solutions or abrasive materials to clean the device ; they could damage the device.

**TIGEX RECOMMENDS :** To avoid these types of disadvantages, clean the heating surface after letting it cool off using a cloth moistened in a 30 ml solution 50 % water and 50 % white vinegar.

2-year guarantee

Please send a copy of your proof of purchase, your name, surname and address on a sheet of paper, and the defective product to:

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